



## Temperature Screening Kit – Setup Guide

1. Unbox equipment and verify all parts are present.
2. Locate battery backup (F) and plug into outlet.
3. Press power button on battery backup (F) and verify that display lights up.
4. Locate Dell PC (A) and Dell PC Power Supply (J).
5. Plug power supply cable into Dell PC (A) and the outlet end to the plug marked PC on the battery backup (F).
6. Plug PoE switch power supply cable (L) into PoE switch (B) and the outlet end into the outlet marked switch on the battery backup (F).
7. Camera assembly (C) will snap onto the tripod (H) via the quick release connector.
8. Plug one end of the 50ft network cable (D) into the camera (C). Plug the opposite end into port 1 of the PoE switch (B).
9. Elevate the tripod (H) to 5ft via the adjustable legs.
10. Plug the 1ft network cable (E) into the back of the Dell PC (A). Plug the opposite end into port 5 of the PoE switch (B).
11. Press the power button on the front of the Dell PC (A). The button will light up when powered on.
12. Plug HDMI cable (G) into monitor (I). Plug the opposite end into Dell PC (A).
13. Plug monitor power cable (K) into bottom of monitor (I). Plug opposite end into the battery backup (F) in the outlet labeled monitor. Press power button.
14. When you see the login screen, please login with the password: HerringTMS!
  - a. *If the above password does not work, please try the following:*
    - i. Type your company name (capitalized) followed by the number 1 and an exclamation mark: *[CompanyName]1!*
    - ii. *E.g. HerringTechnology1!*
15. The camera requires a 30-minute acclimation period before getting accurate readings.
16. Any subjects that alert should be tested with an FDA approved device for verification.
17. For the **Software Instructions**, please go to:
  - a. <https://herringtechnology.com/temperature-monitoring-videos>